




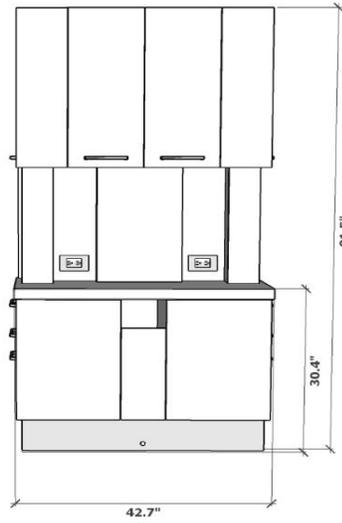

Anthos Perfetta Integrated Treatment Console
Installation Guide
Version 080822
Cover Page

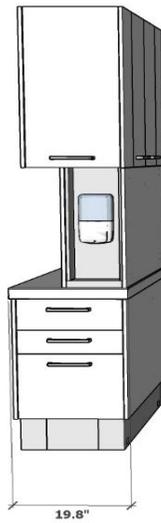

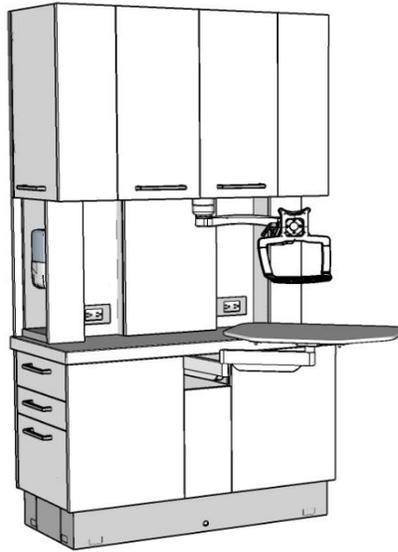


**Anthos Perfetta Integrated Treatment Console
Installation Guide
Version 080822
Installation Sight Prep**

Prior to receiving the APITC's the sight will require the following accomodations:

Location to connect two hospital grade duplex GFCI receptacles. Location to be in the green box shown below. All wiring is to be completed by a licensed electrician.

Location to plug in a task light. Location to be in the green box shown below.

Top Down View of Floor

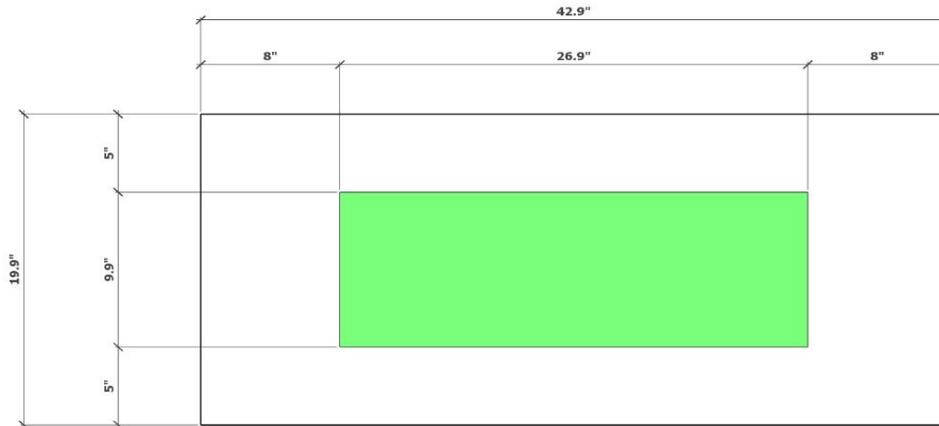



**Anthos Perfetta Integrated Treatment Console
Installation Guide
Version 080822
Product Receiving**

The APITC will arrive at your requested location secured to a pallet that is approximately 49" long, 60" wide, and 50" tall. It will weigh approximately 650 lbs. Sizes and weights will vary based on configurations.

Please inspect pallet and colored wrap for any signs of damage that may have been caused by shippers. Report any damage immediately to your Cefla territory manager.

At installation location carefully unwrap all parts. Please note that if your console includes a floating assistant worksurface it is very thin and is secured between protective cardboard. It is easy to overlook this part.

All provided hardware will be located in the drawers of the base cabinet.




**Anthos Perfetta Integrated Treatment Console
Installation Guide
Version 080822
Assembly**


Step 1 - Set base cabinet with steel toe kick in place. Remove all doors and drawers. Level the base cabinet using a flat head screw driver to adjust the four leveling feet attached to the steel base.

Step 2 - If integrating the assistants floating worksurface complete step 2 otherwise progress to step 3. Set the flat mounting surface on the top of the base cabinet and line up the six holes in the steel plate with the six pre-drilled holes in the base cabinet top. Place t-nut in the underside of each hole and secure with the provided bolts. Set the post attached to the floating worksurface into the hole at the end of the arm.

Step 3 - With chaise holes towards the back of the cabinet set the counter top on top of the base cabinet. If the cabinet is to be free standing the counter will overhang the back side by approximately 10mm. If the cabinet is flush to a wall the counter back will be flush to the cabinet back.

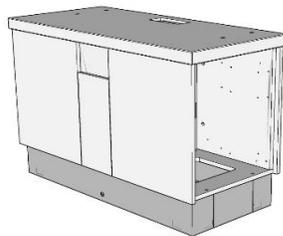


Step 4 - Set middle hutch on top of counter top and line up accent panels of middle hutch and base cabinet. Using the 4 provided 3" Allen drive bolts attach the base cabinet to the counter and the middle hutch. The bolts enter through the base cabinets, go through the counter, and into the four L bracket nuts pre-installed in the middle hutch. Readjust the counter top and tighten all four bolts.

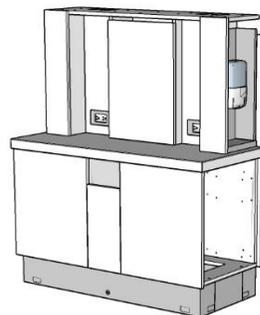



**Anthos Perfetta Integrated Treatment Console
Installation Guide
Version 080822
Assembly**

Step 5 - Set the upper cabinet on top of the middle hutch and line up accent panels. Attach the upper cabinet to the middle hutch using the 6 provided 1.5" Allen drive bolts. The bolts enter through the upper cabinet and go down into the middle hutch.

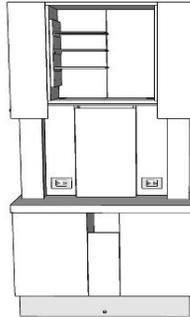

Step 6 - Relevel cabinet and attach to the floor. 10 through holes are available in the steel base to attach to the floor. Distributor is responsible for providing mounting hardware and for properly securing the cabinet to the floor. All federal and local mounting codes must be followed including all seismic codes.

Step 7 - Reattach all doors and drawers. Attach all hardware. Make adjustments as needed to properly level off all doors and drawers.