


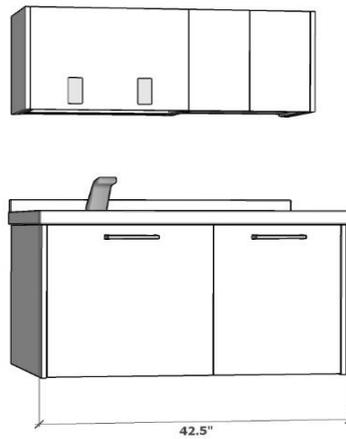


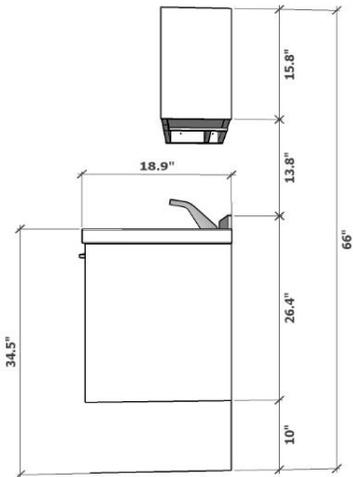

**Anthos Perfetta Side Treatment Console
Installation Guide
Version 080822
Cover Page**

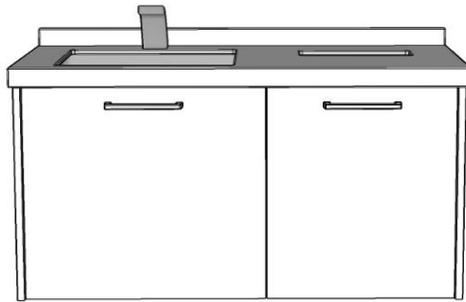

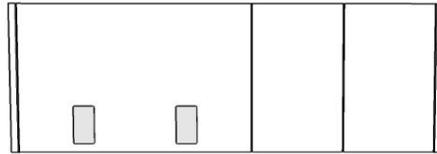

**Anthos Perfetta Side Treatment Console
Installation Guide
Version 080822
Installation Sight Prep**


Prior to receiving the APITC's the sight will require the following accommodations:

2" x 6" in wall supports for base cabinet the entire width of the cabinet. Top of support to be at 33" above ground.

2" x 6" in wall supports for wall cabinet the entire width of the cabinet. Top of support to be at 66" above ground.

Standard plumbing. Location to be on center 18" above ground and 12" from the respective edge of the work surface.

Electrical outlet to plug in the sensed faucet. Location to be on center 24" above ground and between 4" and 10" from respective counter edge.

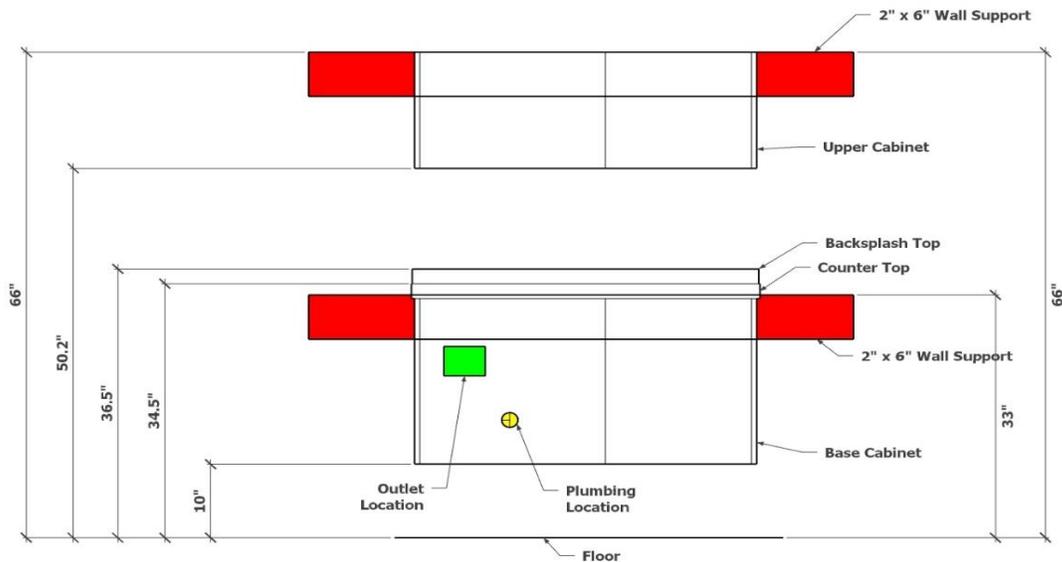



**Anthos Perfetta Side Treatment Console
Installation Guide
Version 080822
Product Receiving**

The APSTC will arrive at your requested location secured to a pallet that is approximately 32" long, 50" wide, and 50" tall. It will weigh approximately 350 lbs. Sizes and weights will vary based on configurations.

Please inspect pallet and colored wrap for any signs of damage that may have been caused by shippers. Report any damage immediately to your Cefla territory manager.

At installation location carefully unwrap all parts.

All provided hardware will be located in the drawers of the base cabinet.



**Anthos Perfetta Side Treatment Console
Installation Guide
Version 080822
Assembly**


Step 1 - Remove all doors and drawers from the cabinets.

Step 2 - Position the base cabinet to the desired location and mount 10" off of the ground. Distributor is to provide all mounting hardware and is to mount all cabinets in accordance with all federal and local regulations including seismic codes.

Step 3 - Attach sink to the underside of the counter top using silicone adhesive and included hardware. When set in place the strainer should be located on the left side of the sink. Silicone is to be provided by distributor or plumber. When attaching clips to the sink avoid overtorquing as this may cause bonding failure.



Step 4 - Set counter on top of base cabinets and secure using silicone adhesive between top of cabinet and bottom of countertop. Make sure the overhang on both sides are equal.

Step 5 - Attach the back splash to the wall using clear silicone adhesive. Center the backsplash prior to applying.

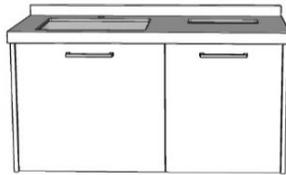

Step 6 - Attach faucet based on directions provide with the faucet. Faucet will plug into outlet inside of base cabinet.

Step 7 - Mount the upper cabinets to the wall in the same fashion as the base cabinets. Height of top of upper cabinet is 66"

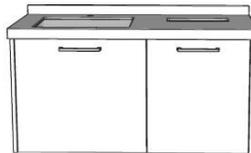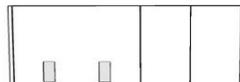

Step 8 - Reattach all doors and drawers. Attach all hardware. Make all final door and drawer adjustments.